

Client Selection Guidelines

The new TherAlign Dragonfly™ TLSO system combines three outstanding clinical components into one solution for flexible postural deficits:

- A comfortable, foam-lined, Velcro-sensitive TheraTogs garment system that grips the skin. The patented composite fabric is elasticized with a stretch bias that favors postural alignment.
- A customizable TheraTogs Dragonfly[™] orthotic component that attaches quickly and easily to the base garment and strapping system for full-trunk support.
- A set of strong, TogRite™ grip-lined elastic straps that operate like muscles to improve alignment and correct neuromotor function at targeted sites.

TherAlign Dragonfly TLSO = A New Generation of Orthoses

If your client has	The Dragonfly TLSO system can:
Hypotonia	✓ Improve postural alignment and spinal positioning for function.
Flexible kyphosis	✓ Shorten and re-train trunk extensor muscles.
Flexible lordosis	✓ Limit trunk movement in the sagittal plane.
Excess pelvic tilt	✓ Improve respiratory capacity by reducing flexible kyphosis.
Diminished pelvic tilt	✓ Reduce postural deviations between thorax and pelvis.
Weak trunk muscles	✓ Reduce pressure on intervertebral disks via intracavitary pressure.
Shoulder protraction	✓ Retract forward shoulders.
Scapular winging	✓ Gently stabilize the scapula on the thorax.
	✓ Provide lightweight unobtrusive trunk support while worn under client's clothing.

Sizing & Ordering

Sizing a TherAlign Dragonfly is a four-step process:

- 1. Take proper measurements as outlined in the TherAlign Sizing Chart.
- 2. Select the proper size for the TheraTogs garment system component.
- 3. Select the proper Dragonfly unit corresponding to spinal measurements.
- 4. Select rib and pelvic strapping length.